Ammonia

Control

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- Young, D.S., Effects of preanalytical variables on clinical laboratory test. AACC Press. Third ed. (2007).

Ammonia

Control

For precision control in ammonia determination

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Nr kat. 1009693

USE

Ammonia Control is designed to be used in the precision control of the **Ammonia** kit from Wiener lab. Refer to the table of the components' assigned values, since they are lot-specific.

PROVIDED REAGENTS

Control level 1: ammonia in normal concentration, in saline solution.

Control level 2: ammonia in pathological concentration, in saline solution.

INSTRUCTIONS FOR USE

The Controls are provided ready to use.

WARNINGS

Controls are for "in vitro" diagnostic use. Do not ingest. Avoid contact with eyes and skin.

Use the reagents according to the working procedures for clinical laboratories.

All reagents and samples must be discarded according to local regulations in force.

STABILITY AND STORAGE INSTRUCTIONS

Provided Reagents: are stable at 2-10°C until the expiration date shown on the bottle.

INSTABILITY OR DETERIORATION OF REAGENTS

The reagents should be clear. Discard if turbidity is observed.

PROCEDURE

The Controls should be used in the same way as an unknown sample, according to the instructions included in the **Ammonia** kit.

WIENER LAB PROVIDES

- 2 x 1 ml (ICat. N° 1999746)

REFERENCES

- Tietz, N. W. (Ed): Fundamentals of Clinical Chemistry, W. B. Saunders Co., Philadelphia, 865 (1982)
- Young, D.S., Effects of preanalytical variables on clinical laboratory test. AACC Press. Third ed. (2007).

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Wiener Laboratorios S.A.I.C.