# HDL Cholesterol

*Calibrator*

Para la calibración de la determinación de HDL-colesterol  
Para a calibração da determinação de HDL-colesterol

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# HDL Cholesterol

## Calibrator

For the calibration of the HDL-cholesterol determination

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Nr kat. 1220232

### USES

**HDL Cholesterol Calibrator** is designed to be used for the calibration of the HDL Cholesterol fast kit from Wiener lab. Check the given value on the label, since it is lot-specific.

### PROVIDED REAGENTS

**Calibrator:** lyophilized human serum containing various types of lipoproteins, including HDL-cholesterol.

### NON-PROVIDED REAGENTS

Distilled or bidistilled water.

### INSTRUCTIONS FOR USE

Reconstitute the vial adding the quantity of distilled or bidistilled water stated on the label. Cap, let stand for 5 minutes and homogenize gently (without stirring) until obtaining a complete dissolution, before use.

### WARNINGS

Reagents are for "in vitro" diagnostic use.

Avoid ingestion and contact with the skin.

This calibrator has been prepared from non-reactive material to HBsAg, HCV and HIV. However, it should be handled as if it were an infectious material.

Use reagents according to the working procedures for clinical laboratories.

The reagents and samples should be discarded according to the local regulations in force.

### STABILITY AND STORAGE INSTRUCTIONS

The Calibrator is stable in refrigerator (2-10°C) until the expiration date shown on the box. After reconstitution, the calibrator is stable for 1 week in refrigerator (2-10°C) or for 1 month frozen (-20°C), avoiding thawing and refreezing.

### INSTABILITY OR DETERIORATION OF REAGENT

The reagent must be clear. Discard if turbidity is present.

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Wiener Laboratorios S.A.I.C.