Homocysteine

Control

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Homocysteine

Control

For precision control of total homocysteine determination

USES

Homocysteine Control has been designed to be used in the quality control of the **Homocysteine** kit.

Check the given value chart for the constituents, since they are lot specific.

Monitoring calibrator results aid in the detection of analytical errors caused by faulty technique, reagent deterioration or instrument problems.

PROVIDED REAGENTS

Control level 1: human serum containing homocysteine normal concentrations.

Control level 3: human serum containing homocysteine pathological concentrations.

INSTRUCTIONS FOR USE

Provided Reagents: ready to use.

WARNINGS

The reagents are for "in vitro" diagnostic use.

Controls have been prepared from non-reactive material for HBsAg, HCV and HIV. All patient samples and controls should be handled as capable of transmitting infectious diseases.

Use the reagents according to the working procedures for clinical laboratories.

Reagents and samples should be discarded according to the local regulations in force.

STABILITY AND STORAGE INSTRUCTIONS

Provided Reagents: stable at 2-10°C until the expiration date stated on the box. Once opened, it is stable for up to 30 days at 2-10°C.

INSTABILITY OR DETERIORATION OF REAGENTS

Reagents are limpid solutions. Discard in case of turbidity.

PROCEDURE

Controls should be used in the same way as an unknown sample, according to the **Homocysteine** instructions for use.

Remove from the refrigerator and swirl gently (vigorous swirling will create unwanted bubbles).

Transfer aliquots of the required size to appropriate sample cups for the instrument being used.

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Wiener Laboratorios S.A.I.C.