# LA Control

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# LA Control

Positive control for lupus anticoagulant

## USE

**LA Control** is designed for use as positive control for LA Confirm and LA Screen.

Refer to the table of assigned values, because they are lot specific.

## PROVIDED REAGENTS

**Control:** citrated human plasma positive for lupus anticoagulant, lyophilized

## NON-PROVIDED REAGENTS

- Wiener lab.'s LA Confirm
- Wiener lab.'s LA Screen

## INSTRUCTIONS FOR USE

- Open the vial carefully to avoid loss of lyophilized material.
- Add 1.0 ml of distilled water.
- Cover and mix by gentle rotation until complete reconstitution, avoiding foam formation.
- Dissolve for 10 minutes at room temperature before use.

## WARNINGS

Reagents are for "in vitro" diagnostic use.

The Control was prepared from non-reactive material for HBsAg, antibodies to HCV and antibodies to HIV 1+2. Nonetheless, the calibrator and all blood samples should be handled as if capable of transmitting infections.

Use the reagents according to the working procedures for clinical laboratories.

The reagents and samples should be discarded according to the local regulations in force.

## STABILITY AND STORAGE INSTRUCTIONS

Stable in refrigerator (2-10°C) until the expiration date stated on the box.

After reconstitution it is stable for 3 hours refrigerated (2-10°C) or for 1 month frozen (-20°C). It is recommended to freeze only once. Keep the bottles tightly closed.

## PROCEDURE

Reconstituted LA Control should be treated as an unknown sample by following the instructions of the test in use.

Screen. When LA Confirm is used, the results are lower or normal.

## PROCEDURE LIMITATIONS

Failures in the reconstitution or preservation may cause erroneous results.

Refer to the Procedure Limitations in the package insert of the kit in use.

The values obtained should be considered only as a guide, depending on the analyzer or coagulometer used.

## WIENER LAB PROVIDES

2 x 1 mL (Cat. N° 1937006)

## REFERENCES

- T. Exner, K. A. Rickard, H. Kronenberg: A Sensitive Test Demonstrating Lupus Anticoagulant and its Behavioural Patterns. *Brit. J. Haem.* 40 (1978); 143.
- E. Rosner, R. Pauzner, A. Lusky, M. Modan, A. Many: Detection and Quantitative Evaluation of Lupus Circulating Anticoagulant Activity. *Thrombos. Haemostas.* 57 (1987); 144.
- D. A. Triplett: Screening for the Lupus Anticoagulant. *Research in Clin. and Lab.* 19 (1989); 379.

## INTERPRETATION OF RESULTS

**LA Control** should show abnormal test results with LA

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Wiener Laboratorios S.A.I.C.