Microalbúmina

Calibrador

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Microalbúmina

Calibrador

For the calibration of microalbuminuria determination by immunoturbidimetric method

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Nr kat. 1913266

USES

Microalbúmina Calibrador has been designed to be used with Wiener lab's **Microalbúmina Turbitest AA**. Check the value stated in the label, since it is lot-specific. That value has been standardized with the European Reference Material, ERM-DA470 (BCR-470) - IRMM (Institute for Reference Materials and Measurements).

PROVIDED REAGENTS

Calibrator: human albumin solution with the appropriate preservatives.

INSTRUCTIONS FOR USE

Calibrator: ready to use.

Before testing, invert the vial several times avoiding foam formation. Dispense the Calibrator volume required to be used. Do not return the remaining material into the original vial.

WARNINGS

The reagent is for "in vitro" diagnostic use.

Avoid ingestion and contact with the skin.

This Calibrator has been prepared from non-reactive material to hepatitis B surface antigen (HBsAg) and antibodies to hepatitis C virus (HCV) and human immunodeficiency virus (HIV). However, the Calibrator and all samples should be handled as infectious material.

Use the reagents according to the working procedures for clinical laboratories.

The reagents and samples should be discarded according to the local regulations in force.

STABILITY AND STORAGE INSTRUCTIONS

The unopened Calibrator is stable at 2-10°C until the expiration date stated on the box.

Once opened, it is stable for up to 4 weeks at 2-10°C.

Tightly cap the vial and store it at 2-10°C after use. Avoid any action that may result in Calibrator contamination.

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Wiener Laboratorios S.A.I.C.