Microalbúmina

Control - 2 niveles

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For precision control

USES

Microalbúmina Control has been designed to control the precision of Wiener lab's **Microalbúmina Turbitest AA**. Check the given values chart for each level. The values have been standardized with the European Reference Material, ERM-DA470 (BCR-470) - IRMM (Institute for Reference Materials and Measurements).

- Sacks et al - Clin. Chem. 48:436 (2002).
- Mogensen, CE - J. Intern. Med. 254:45 (2003).
- American Diabetes Association: Diabetic Nephropathy. Diabetes Care (Suppl 1) 26:S94 (2003).

PROVIDED REAGENTS

Control Level 1: human albumin solution containing albumin levels ranging from 20 to 60 mg/l.

Control Level 2: human albumin solution containing albumin levels ranging from 80 to 160 mg/l.

INSTRUCTIONS FOR USE

Provided Reagents: ready to use.

Before testing, invert the vial several times avoiding foam formation. Dispense the required Control volume to be used. Do not replace the already used material into the original vial.

WARNINGS

The reagent is for "in vitro" diagnostic use.

Avoid ingestion and contact with the skin.

The Controls have been prepared from non-reactive material to hepatitis B surface antigen (HBsAg) and antibodies to hepatitis C virus (HCV) and human immunodeficiency virus (HIV). However, the Controls and all the samples should be handled as infectious material.

Use the reagents according to the working procedures for clinical laboratories.

The reagents and samples should be discarded according to the local regulations in force.

STABILITY AND STORAGE INSTRUCTIONS

The unopened Control is stable in refrigerator (2-10°C) until the expiration date shown on the box.

Once opened, it is stable for 4 weeks in refrigerator (2-10°C).

Tightly cap the vial and store it in refrigerator after use. Avoid any action that may result in the Control contamination.

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Wiener Laboratorios S.A.I.C.