Carbon Dioxide

Control

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2 x 1 mL (Cód. 1999750)

REFERÊNCIAS

- Tietz, N. W. (Ed): Fundamentals of Clinical Chemistry, W. B. Saunders Co., Philadelphia, 865 (1982)
- Young, D.S., Effects of preanalytical variables on clinical laboratory test. AACC Press. Third ed. (2007).

Carbon Dioxide

Control

For precision control in carbon dioxide determination

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Nr kat. 1009671

USE

Carbon Dioxide Control is designed to be used in the precision control of the **Carbon Dioxide** kit from Wiener lab. Refer to the table of values assigned to the components, since they are lot-specific.

PROVIDED REAGENTS

Control level 1: sodium bicarbonate in normal levels, in saline with sodium azide as preservative.

Control level 2: sodium bicarbonate in pathological levels, in saline with sodium azide as preservative.

INSTRUCTIONS FOR USE

The controls are provided ready to use.

WARNINGS

Controls are for "in vitro" diagnostic use. Do not ingest. Avoid contact with eyes and skin.

Use the reagents according to the working procedures for clinical laboratories.

All reagents and samples must be discarded according to local regulations in force.

STABILITY AND STORAGE INSTRUCTIONS

Provided Reagents: are stable at 2-10°C until the expiration date shown on the bottle.

INSTABILITY OR DETERIORATION OF REAGENTS

The reagents should be clear. Discard if turbidity is observed.

PROCEDURE

The Controls should be used in the same way that an unknown sample, according to the instructions for use accompanying the **Carbon Dioxide** kit.

WIENER LAB PROVIDES

2 x 1 mL (Cat. N° 1999750)

REFERENCES

- Tietz, N. W. (Ed): Fundamentals of Clinical Chemistry, W. B. Saunders Co., Philadelphia, 865 (1982)
- Young, D.S., Effects of preanalytical variables on clinical laboratory test. AACC Press. Third ed. (2007).

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Wiener Laboratorios S.A.I.C.