Ceruloplasmin

Calibrator

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- 1 x 1 ml (Cód. 1999748).

REFERÊNCIAS

- Dati, F et al. - Proteins-Laboratory testing and clinical use, 2005.
- Tietz Textbook of Clinical Chemistry - Burtis, C.; Ashwood, E. (5ª Edition) WB Saunders, 2001.
- Young, D.S. - "Effects of Drugs on Clinical Laboratory Tests", AACC Press, 5th ed., 2000.

Ceruloplasmin

Calibrator

For the calibration of the ceruloplasmin determination by immunoturbidimetric methods

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Nr kat. 1999748

USES

Ceruloplasmin Calibrator Turbitest AA has been designed for the calibration of Wiener lab.'s **Ceruloplasmin Turbitest AA**. Since it is lot-specific, check the given value on the label.

PROVIDED REAGENTS

Calibrator: inactivated human serum containing appropriate preservatives.

INSTRUCTIONS FOR USE

Calibrator: ready to use.

Before testing, gently invert the vial several times avoiding the formation of foam. Remove the amount of calibrator to be used. Do not return the used material to the original vial.

WARNINGS

Reagents are for "in vitro" diagnostic use. Avoid ingestion and contact with the skin.

This calibrator has been prepared from non-reactive material to HBsAg, HCV and HIV. However, it should be handled as if it were an infectious material.

Use reagents according to the working procedures for clinical laboratories.

The reagents and samples should be discarded according to the local regulations in force.

STABILITY AND STORAGE INSTRUCTIONS

Calibrator: is stable at 2-10°C until the expiration date stated on the box.

Once reconstituted it is stable for 4 weeks at 2-10°C. Keep the bottle tightly closed after use. The Calibrator is stable for 3 months at -20°C. It is recommended that the calibrator is aliquoted and frozen immediately after using. Thawing is recommended only once. Avoid contamination.

WIENER LAB. PROVIDES

- 1 x 1 ml (Cat. N° 1999748).

REFERENCES

- Dati, F et al. - Proteins-Laboratory testing and clinical use, 2005.
- Tietz Textbook of Clinical Chemistry - Burtis, C.; Ashwood, E. (5^o Edition) WB Saunders, 2001.
- Young, D.S. - "Effects of Drugs on Clinical Laboratory Tests", AACC Press, 5th ed., 2000.

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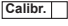
- 1 x 1 ml (Nr kat. 1999748)

ŹRÓDŁA

- Dati, F et al. - Proteins-Laboratory testing and clinical use, 2005.
- Young, D.S. - "Effects of Drugs on Clinical Laboratory Tests", AACC Press, 5th ed., 2000.

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Wiener Laboratorios S.A.I.C.